

Quick. Simple. Mess- Free.

Packaged in a convenient, all-in-one system, the Varnish Pen replaces sticky foil packets, varnish pools and brush sticks. Say goodbye to double-dipping with a mess-free applicator that sends varnish from brush-to-teeth with a quick twist and easy press.

Tips

For best results, thoroughly remove biofilm and eliminate excessive moisture. Apply a thin film of varnish to all tooth surfaces and allow to dry for 10 seconds. Varnish will quickly set with moisture from saliva. Patient can consume soft foods and cold drinks immediately following treatment but should avoid hot drinks, hard food, alcohol (including mouth wash), brushing and flossing for 6 hours.

Easy Appplication

Step 1: Twist

- Twist the color dial in direction of arrow while applying slight downward pressure to unlock varnish.
- Turn brush to desired position.

Step 2: Press

• Press tube to squeeze varnish through bristles to teeth. Just release to stop flow. varnishpen

2

For Single Use Only

Warning: Avoid contact with eyes and skin. If contact occurs, flush immediately with plenty of water and consult a physician. Keep out of reach of children and do not store together with food. Visit www.youngdental.com for Safety Data Sheet. For emergencies contact INFOTRAC at 1-800-535-5053.

Indications:

Varnish is a fluoride-containing preparation for the treatment of dentinal hypersensitivity and for the reduction of post-operative sensitivity.

Dosage:

For the treatment of dentinal hypersensitivity, two applications at several days' interval are sufficient in most cases. The fluoride content of the varnish is dosed in such a way that neither acute nor chronic side effects are to be expected if applied according to instructions. 1mL of varnish contains 50mg of sodium fluoride, equivalent to 22.6 mg fluoride ion. Varnish is sweetened with sucralose and xylitol.

Contraindications:

Ulcerative gingivitis and stomatitis.

Interactions:

When varnish is applied, other fluoride preparations such as fluoride gels should not be administered during the same day. The routine use of fluoride tablets should be interrupted for several days after treatment.

Adverse Reactions:

Edematous swellings have been reported only in rare instances in some fluoride varnish products, especially after application to extensive surfaces. Dyspnea, although extremely rare, has occurred in asthmatic children. Nausea has been reported when extensive applications have been made. If required, varnish film can be removed with a thorough brushing.

Refer to Safety Data Sheet for additional information. For emergencies, contact INFOTRAC at 800-535-5053 Store at room temperature. Highly flammable.

Meet the Future

The **NEW** Way to Apply Fluoride Varnish is Here!





13705 SHORELINE COURT EAST

• EARTH CITY, M0 63045 USA • 800-325-1881 • FAX 314-344-0021 • WWW YOUNGDENTAL COM • MADE IN USA • 295856C

US Patent No. 9,427,079 US Patent No. D785,797



